

Product description

Active is a sterile, wound dressing, coated with honey-based hydrogel. This dressing contains: 30% medical grade honey, acrylic polymer gel and water with a polyurethane film backing.

The product creates a moist wound healing environment and can absorb over 700% fluid in 24 hours. A moist wound environment stimulates the wound healing process by facilitating autolysis of necrotic and devitalized material, aids wound bed preparation and promotes epithelialization.

The dressing can be worn during showering, reduces rubbing and friction, cushions bony areas, conforms to contoured areas, and moves with the patient's body.

Intended purpose & indications

Active is for minor burns, superficial cuts, lacerations and abrasions and other small wounds. Patients who are known to be sensitive or allergic to the dressing or any of its components should not use it. The product is for single use only. Active can be used by healthcare professionals and laypersons.

Precautions & Warnings

Do not use it if the primary packaging is damaged, contaminated or expired.

Due to autolytic debridement the wound may appear to increase in size after the first dressing changes. This is normal and occurs as the devitalised tissue is removed, preparing the wound bed for healing.

If the wound becomes swollen, reddened or painful, or does not start to heal within a few days; seek medical advice.

Inspect the wound and the surrounding tissue regularly, as per local protocols.

In the event of intolerance to the dressing or one of its components, remove the dressing using appropriate techniques and clean the area carefully.

Instructions for use

1. Cleanse the wound as per local protocol.
2. Remove the sterile product from the pouch.
3. Remove backing and apply the dressing to the wound.
4. Apply the dressing to a surface that is clean and not too greasy; otherwise it may not adhere properly.
5. Ensure the adhesive part is in contact with the surrounding intact skin in order to achieve a proper closure.

Frequency of change

Active can stay up to 7 days on the wound, depending on the state of the wound and amount of exudate produced. In the event the product has become too dry, complete and gentle removal of the product may be aided by irrigation.

Storage and transport conditions

Store at room temperature between 5°C and 25C (41°-77F). Store in a dry place, away from direct sunlight.

Incident reporting

Please notify the manufacturer and the competent authority of your country in case of any serious incident, i.e. incident that directly or indirectly led, might have led or might lead to death of a patient, user or other person, temporary or permanent serious deterioration of patient's, user's or other person's state of health, serious public health threat.

Product disposal

Dispose the used product as per local protocol.

Date of revision of the text: 06-10-2025

IFU's identifier: IFU-A-XX-A

Theo Manufacturing BV, October 2025, v5, IFU-A-A